

Suction Toothbrush For Oral Hygiene

Brush head with all around bristles to ensure thorough cleansing, bristles made from medical-grade silicone soft enough to avoid mucosal injury.

Three holes on the head to ensure enough suction, irrigation and prevent risks of aspiration.


The wide and grip design provides better comfort and control to the operator.

Brush handle made from K-resin, transparency allows observing the situation of fluid in the oral cavity, rigid enough to avoid being bitten off.

Vacuum vent hole offers intermittent or continuous suction control.

The Suction Lumen to be connected onto suction tube to suction fluid in oral cavity. The vacuum aspiration range should be set up between 40kpa - 53.3kpa.

Irrigation lumen can be connected onto infusion set to provide enough saline or mouth water to rinse.

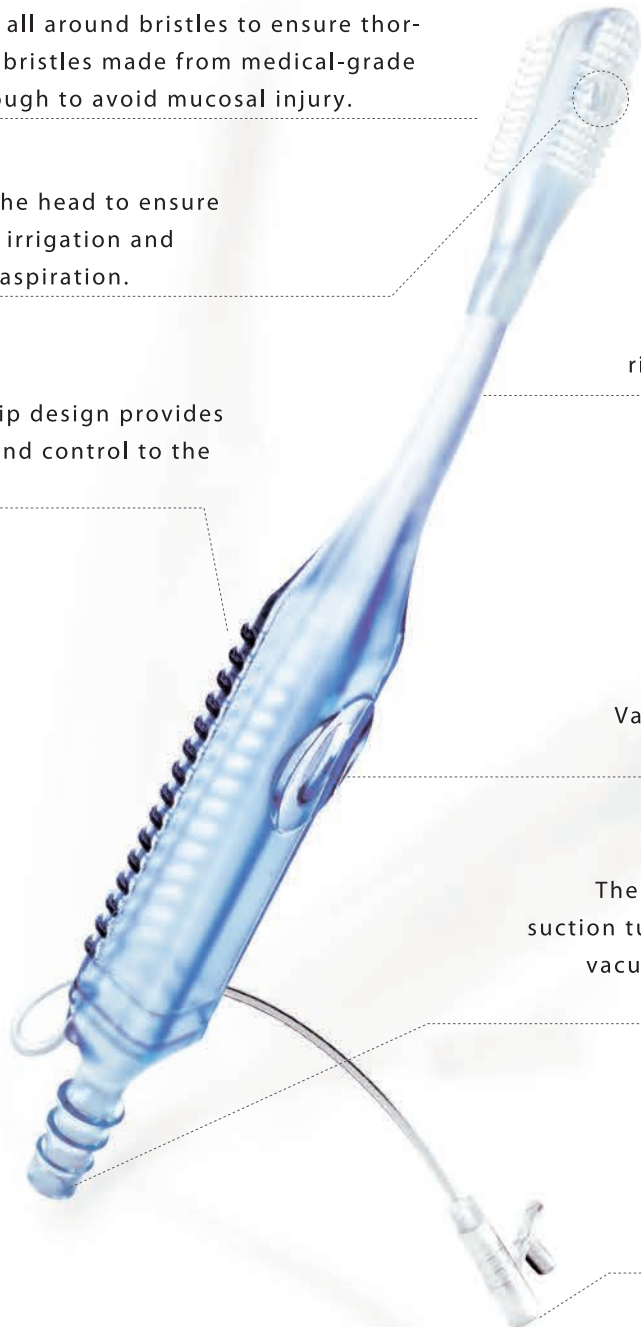

WELL LEAD MEDICAL CO., LTD.

Add: 47 Guomao Avenue South, Panyu Guangzhou, China
Marketing&Trading Office

Add: RM 805, No 55, 21 Long, Pusan Road, Shanghai 200125, China

Tel: +86-021-53897623 Fax: +86-021-53897615 Post Code: 200127

www.wellead.com.cn e-mail: info@wellead.com

wellead®

Oral hygiene is now recognized as a fundamental factor to avoid aspiration pneumonia and other infections associated with mechanical ventilation .



Ventilator associated pneumonia (VAP) is the leading cause of morbidity and mortality in intensive care units.



CUSTOMIZABLE KIT to perform oral care lower the pulmonary infection and prevent VAP.



Caution

- Recommended suction pressure: 40~53.5Kpa
- Suctioning intermittently during irrigation
- Flow rate of infusion set should be around 80 Drops/min

Size	Cat. No.
Adult	N04A010210
Oral care kit	N04A020110

* Oral care kit can be customized